
noah
LINK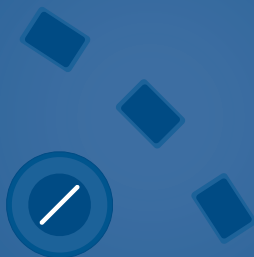

Hearing Instrument
Manufacturers'
Software Association

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7-26-66403/05
2010-503300 (R08)

NOAHlink Users Manual

NOAHlink is manufactured for HIMSA by:

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Trademarks

NOAHlink is a trademark of HIMSA A/S.

Windows is a registered trademark of Microsoft Corporation.

MPEG Layer-3 audio decoding technology licensed from Fraunhofer IIS and Thomson.

Copyright

This manual is copyrighted by HIMSA A/S with all rights reserved. Under copyright laws, this manual cannot be reproduced in any form without written permission of HIMSA A/S.

© 2001 by HIMSA A/S. All rights reserved.

Information in this document is subject to change without notice.

Contents

1	Introduction	5
1.1	ABOUT THIS MANUAL	5
2	Safety Issues	6
2.1	MANUFACTURER RESPONSIBILITY	6
2.2	SYMBOLS	6
2.2.1	NOAHlink Symbols.....	6
2.2.2	Charger Unit Symbols	7
2.3	WARNING NOTES.....	7
2.3.1	NOAHlink Warning Notes	7
2.3.2	Charger Unit Warning Notes.....	9
3	Description	10
3.1	STANDARD ELEMENTS.....	10
3.2	OPTIONAL ACCESSORIES	11
3.3	OVERVIEW	11
4	Maintenance	12
4.1	UNPACKING AND INSPECTION	12
4.2	STORAGE AND SHIPMENT	12
4.3	CARE AND MAINTENANCE.....	13
4.4	SERVICE AND REPAIR	13
5	Installation	14
6	Battery Usage	16
6.1	ALKALINE BATTERY.....	16
6.2	RECHARGEABLE BATTERY	17
6.2.1	Discharging and Recharging.....	18
7	Using NOAHlink	19
8	Configuring NOAHlink	22
8.1	GENERAL APPLICATIONS.....	24
8.1.1	Connecting to NOAHlink.....	24
8.1.2	Setting Power, Battery and Language Options.....	26
8.2	ADVANCED APPLICATIONS	28
8.2.1	Displaying Errors in NOAHlink and Bluetooth.....	28
8.2.2	Status Settings and User Preferences	29
8.2.3	Testing Data Transfer Rates.....	31
8.2.4	Version Numbers and Kernel Upgrade	32
8.2.5	Viewing and Setting the Log	33

A	NOAHlink Troubleshooting	35
B	Technical Specifications	36
B.1	NOAHLINK UNIT.....	36
B.1.1	Battery	36
B.1.2	Dimensions and Weight.....	36
B.1.3	Patient Safety.....	36
B.2	CHARGER UNIT	37
B.2.1	Power Supply	37
B.2.2	Dimensions and Weight.....	37
B.3	POWER ADAPTOR	37
B.3.1	Power Supply	37
B.4	TRANSPORT AND STORAGE	37
B.5	OPERATING ENVIRONMENT	37
B.6	STANDARD AND OPTIONAL ACCESSORIES	37

1 Introduction

Your NOAHlink is a wireless Bluetooth communication device optimized for use with NOAH 3. NOAHlink allows you to program hearing instruments with ease, both in your office and on location. You can use rechargeable or alkaline batteries to power your unit.

Figure 1: NOAHlink, Neck String and Charger Cradle

1.1 About this Manual

This manual provides a detailed description of the following NOAHlink procedures:

- installation
- daily use
- charging/discharging
- configuration

In addition, the following relevant areas are addressed:

- safety
- maintenance
- technical specifications

2 Safety Issues

2.1 Manufacturer Responsibility

The manufacturer is to be considered responsible for effects on safety, reliability, and performance of the equipment **ONLY IF:**

- all assembly operations, extensions, re-adjustments, modifications or repairs are carried out by the equipment manufacturer or personnel authorized by the manufacturer
- the electrical installation to which the equipment is connected complies with EN/IEC requirements
- the programming cable and connection to the hearing instrument comply with the requirements in Section B.1.3 – Patient Safety
- the equipment is used in accordance with the instructions for use

2.2 Symbols

2.2.1 NOAHlink Symbols

SYMBOL	DESCRIPTION
	NOAHlink carries this symbol to indicate compliance with Type BF of EN 60601-1.
	NOAHlink carries this symbol when it is important that the user refer to associated warnings in this guide.
	NOAHlink and this manual are CE-marked according to the Medical Devices Directive 93/42/EEC.
	Classified with respect to electrical shock, fire, mechanical and other specified hazards in accordance with UL2601-1 and CAN/CSA-C22.2 NO 601.1-90.
	NOAHlink carries this symbol to identify the positioning of the battery inside the battery compartment.
	NOAHlink carries this symbol to indicate that, in France, it is only permitted to use the device indoors.

SYMBOL	DESCRIPTION
FCC	<p>This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions:</p> <ol style="list-style-type: none"> 1. This device may not cause harmful interference 2. This device must accept any interference received, including interference that may cause undesired operation. <p>Refer to Notes 5 through 7, Section 2.3.1, for more details.</p>
IC	<p>The term "IC:" before the equipment certification number only signifies that the Industry Canada technical specifications were met.</p>

2.2.2 Charger Unit Symbols

SYMBOL	DESCRIPTION
	<p>The charger unit carries this symbol when it is important that the user refer to associated warnings in this guide.</p>
	<p>The charger unit is CE-marked according to the EMC Directive 89/336/EEC.</p>
	<p>The charger unit carries this symbol to indicate that it is suitable for direct current only.</p>

2.3 Warning Notes

IMPORTANT: Keep your unit away from liquids.
Do not allow moisture inside your unit.

2.3.1 NOAHlink Warning Notes

SYMBOL	DESCRIPTION
	<p>Place the battery as indicated in the battery compartment. Use only rechargeable batteries when NOAHlink is placed in the charger unit. If you are using alkaline batteries, do not attempt to charge your NOAHlink. Your alkaline batteries may be damaged and leak, and this may in turn cause damage to your NOAHlink unit. Damage caused by such misuse will not be covered by the NOAHlink warranty.</p>

Note 1: There are no user-serviceable parts inside the NOAHlink cabinets. For the sake of safety, and in order not to void the warranty, the cabinets should only be opened and serviced by authorized service personnel. In case of defects, please make a detailed description of the defect(s) and contact your supplier. Do not use a defective instrument.

Note 2: Do not use the instrument in the presence of flammable anesthetics (gases).

Note 3: NOAHlink may be disposed of as normal electronic waste, according to local regulations. Please investigate local regulations concerning the disposal of rechargeable and alkaline batteries.

Note 4: Unwanted noise may occur if NOAHlink is exposed to a strong radio field. Such noise may interfere with the process of fitting a hearing instrument. Many types of electrical devices, e.g. mobile telephones, may generate radio fields. We recommend that the use of such devices in the vicinity of NOAHlink is restricted as much as possible.

Note 5: Changes or modifications not expressly approved by the manufacturer could void the user's authority to operate the equipment.

Note 6: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Note 7: For use in Canada: To prevent radio interference to the licensed service, this device is intended to be operated indoors and away from windows to provide maximum shielding. Equipment (or its transmit antenna) that is installed outdoors is subject to licensing.

2.3.2 Charger Unit Warning Notes

SYMBOL	DESCRIPTION
	<p>Do not use NOAHlink with patients while it is placed in the charger unit. Use only rechargeable batteries when NOAHlink is placed in the charger unit. If you are using alkaline batteries, do not attempt to charge your NOAHlink. Your alkaline batteries may be damaged and leak, and this may in turn cause damage to your NOAHlink unit. Damage caused by such misuse will not be covered by the NOAHlink warranty.</p>

Note 1: The charger unit should be kept away from the patient area.

Note 2: There are no user-serviceable parts inside the charger unit cabinets. For the sake of safety, and in order not to void the warranty, the cabinets should only be opened and serviced by authorized service personnel. In case of defects, please make a detailed description of the defect(s) and contact your supplier. Do not use a defective instrument.

Note 3: The charger unit may be disposed of as normal electronic waste, according to local regulations.

3 Description

Your complete NOAHlink system consists of both standard elements and optional accessories.

Figure 2: NOAHlink, Neck String and Cables

3.1 Standard Elements

The following standard elements comprise your NOAHlink system:

- NOAHlink unit
- Charger cradle
- Rechargeable battery
- Power adaptor
- Users manual
- Installation disk
- Name labels
- Neck string
- Bluetooth Ezurio

3.2 Optional Accessories

You can obtain the following optional accessories from your distributor:

- Spare rechargeable batteries
- Spare neck strings
- Soft-sided case to hold NOAHlink, cables and spare batteries

3.3 Overview

The NOAHlink hardware serves as a standardized interface between a PC and programmable hearing instruments – or a remote control for programmable hearing instruments.

The top of the NOAHlink unit contains two connectors (6-pole mini-DIN) for the cables to the programmable hearing instruments (or remote control). This allows both a left and a right hearing instrument to be programmed simultaneously.

The PC sends information about the communication protocol (for example, communication type, voltage levels, clock frequency, etc.) to NOAHlink, and asks NOAHlink to read data from, or write data to, the programmable hearing instrument. The PC software for programming the hearing instrument is made by the hearing instrument manufacturer.

The cables for connecting hearing instruments to NOAHlink are supplied by the hearing instrument manufacturer.

4 Maintenance

4.1 Unpacking and Inspection

NOAHlink is shipped in its own custom-made carton.

We recommend that you keep this carton in case you need to return NOAHlink to your local distributor for service or repair.

You should notify your local distributor in the following circumstances:

- The packaging on your NOAHlink is damaged (contact your carrier also)
- Your NOAHlink is damaged
- Your NOAHlink package does not contain all the standard elements listed in Section 3.1
- Your power adaptor is not the correct country version
- Your NOAHlink fails to function

NOTE: If your NOAHlink needs to be repaired or replaced in a case in which the carrier is liable, the distributor will arrange for this without waiting for the claim against the carrier to be settled.

4.2 Storage and Shipment

Always pack your NOAHlink with great care when storing or shipping the instrument. Use the original carton if you need to return NOAHlink to your local distributor for repair.

Remove the battery when storing or shipping your NOAHlink.

Always store your NOAHlink in a clean, dry environment. Ensure the power has been turned off when the instrument is placed in an enclosed container.

Remember to ensure the following when shipping your NOAHlink:

- Seal the shipping container securely
- Mark the container Fragile to ensure careful handling
- In any correspondence, refer to your NOAHlink by serial number

4.3 Care and Maintenance

Your NOAHlink and charger unit should be kept clean and free from dust. Use a soft, slightly damp cloth with a small amount of detergent to clean the unit and the cradle.

IMPORTANT: Keep your unit away from liquids.

Do not allow moisture inside the unit.

4.4 Service and Repair

Service and repair of electro-medical equipment should only be carried out by the equipment manufacturer or by authorized representatives.

The manufacturer reserves the right to disclaim all responsibility for the operating safety, reliability and performance of equipment serviced or repaired by other parties. Following repair, the safety of all equipment should be verified by a qualified electronics engineer.

5 Installation

1. Select one of the following options:
 - a. **Use the Ezurio Bluetooth device supplied with NOAHlink:**
 - Do NOT attach the Ezurio device until advised to in the following steps.
 - b. **Use an ALREADY installed Bluetooth device other than the Ezurio device supplied with NOAHlink:**
 - In the following steps, use your own Bluetooth device instead of Ezurio.
2. Place a rechargeable/alkaline battery in NOAHlink. Refer to Chapter 6 for further details. (**NOTE:** If you are using a rechargeable battery, you will first need to charge it.)
3. Remove NOAHlink from its charger cradle (if you have placed it there while charging the battery).
4. Place your NOAHlink unit not more than five meters from your PC to ensure adequate wireless communication.
5. Turn on your NOAHlink. Turn on your PC.
6. An installation CD is provided with NOAHlink. Put this into your disk drive. If the program does not begin running straightaway, use Windows Explorer to locate the **Installation** folder and double-click on **Setup.exe**.
7. Select your setup language and click **Next**.
8. After preparation completes, click **Next**.
9. The **Install Shield Wizard Complete** window appears. Click **Finish**.
10. Follow the guidelines on the screen and plug your Bluetooth device into your PC. Position the Bluetooth device at PC height, and vertically, to ensure optimum performance.
11. The **Found New Hardware Wizard** may appear. If so, click **Next** to install the software automatically.

12. Ensure that NOAHlink is still turned on. (If NOAHlink remains idle for more than 30 minutes, it will automatically turn off.)
13. The **NOAHlink Pairing Wizard** window appears. Press **Search** to search for NOAHlink devices.
14. The program searches for your NOAHlink. When a list of devices appears, select the NOAHlink you wish to use. Click **Pair NOAHlink**.
15. A window appears stating **The NOAHlink device was successfully paired**. Click **OK**.
16. Refer to Chapter 8 in order to configure your NOAHlink unit.
17. **NOTE:** If you have more than one NOAHlink unit, it is recommended that you attach one of the name labels that are included in your NOAHlink package to the back of your NOAHlink and write a suitable name on this label. Refer to Section 8.1.1 for details on naming NOAHlink via your PC.
18. Refer to the NOAHlink site at www.himsa.com for:
 - a list of fitting modules that support NOAHlink. These modules can use NOAHlink for programming hearing instruments.
 - support and troubleshooting

6 Battery Usage

When installing your NOAHlink, you can use either rechargeable or alkaline batteries. This section describes the procedure to use for each battery type.

NOTE: It is recommended that you keep alkaline batteries available when using your NOAHlink in case you forget to recharge your rechargeable battery and it runs down during a patient session.

NOAHlink is powered by battery – rechargeable or alkaline. The battery is placed in the compartment at the bottom of the unit, as illustrated below:

Figure 3: NOAHlink and Battery Lid

6.1 Alkaline Battery

Simply open the battery compartment of your NOAHlink unit and place one alkaline AA (R6) 1.5V battery inside in the correct position by matching the symbols on the battery to those on NOAHlink.

6.2 Rechargeable Battery

The rechargeable battery is a NiMH battery.

IMPORTANT: You will first need to charge the rechargeable battery (described below). This takes approximately 13 – 15 hours. Thus, you will need to charge the battery overnight before you can install NOAHlink.

Figure 4: NOAHlink, Charger Cradle and Power Adaptor

The following describes how to charge your rechargeable battery:

- Open the battery compartment of your NOAHlink unit. Place the rechargeable battery in your unit in the correct direction by matching the symbols on the battery to those on NOAHlink.
- Place your NOAHlink in its charger cradle, matching the front of the cradle to the front of your NOAHlink.
- Attach the charger cradle to the power adaptor and plug this into a socket. Your battery automatically begins to charge and a green light is displayed. This procedure takes approximately 13 – 15 hours.

6.2.1 Discharging and Recharging

Your rechargeable battery – a NiMH battery – automatically charges whenever it is placed in the charger cradle. However, approximately once every three months, it is recommended that you completely discharge, and then recharge, your battery. This ensures that the rechargeable battery is always working to its fullest capacity, and also helps to extend its lifetime.

NOTE: The entire process will take approximately 15 – 17 hours.

You will be unable to turn on your NOAHlink while charging or discharging the unit's battery.

IMPORTANT: If you are using alkaline batteries, do not attempt to charge your NOAHlink. If you are using alkaline batteries, do not attempt to charge your NOAHlink. Your alkaline batteries may be damaged and leak, and this may in turn cause damage to your NOAHlink unit. Damage caused by such misuse will not be covered by the NOAHlink warranty.

The following describes the discharging and recharging procedure:

- Place your NOAHlink unit in its charger cradle, matching the front of the cradle to the front of your unit.
- Attach the charger cradle to the power adaptor and plug this into a socket.
- Press the discharging button on the charger cradle. A red light at the front of the unit remains lit while discharging takes place.
- As soon as discharging is complete, the charger cradle automatically begins to recharge your NOAHlink and a green light is displayed.

7 Using NOAHlink

The following procedure describes how to use your NOAHlink to program a hearing instrument:

WARNING: Always remove NOAHlink from its charger cradle before you begin to program a hearing instrument, as the unit switches off automatically when placed in the charger.

Always keep the charger cradle away from the patient area.

Figure 5: Using NOAHlink to program Hearing Instruments

- Switch on your NOAHlink by pressing the button at the front of the unit until the middle light displays. A self-test occurs every time you do this. The following table explains the different LED (light-emitting diode) displays that may occur during the self-test, depending on the power status of the battery:

POWER STATUS	LED DISPLAY
OK	Continuous green light for five seconds
Low (battery low)	Flashing yellow/orange light for five seconds
Error	Flashing red light for 30 seconds, then NOAHlink powers off

Once the self-test has been completed, your NOAHlink will display different LEDs depending on:

- the power status
- whether the unit is communicating with your PC

The following table outlines the different possibilities:

COMMUNICATING WITH PC	POWER STATUS	LED DISPLAY
Yes	OK	Quickly flashing green light
	Low (battery low)	Quickly flashing yellow/orange light
No	OK	Flashing green light every three seconds
	Low (battery low)	Flashing yellow/orange light every three seconds

- Attach the cable/s (recommended by the hearing instrument manufacturers) for the programmable hearing instrument/s to the connector/s at the top of the NOAHlink unit.
- Connect the ends of the neck string to the attachment points of your NOAHlink according to the color of their plastic connections (red and blue). Hang the NOAHlink from the neck of your patient with this string.

OR:

Attach one plastic attachment button of the neck string to your NOAHlink. Place the string around the neck of your patient and attach the remaining neck string to the empty attachment point on your NOAHlink.

- Adjust the length of the neck string if necessary to ensure a comfortable fit on the patient.
- Connect NOAHlink to the hearing instruments via the cables attached to the top of the unit.
- Ensure the patient is not more than five meters from your PC.
- Start the fitting module on your PC and refer to this program for details on programming the hearing instruments.
- When programming a hearing instrument the LED on your NOAHlink, nearest the ear containing the instrument, flashes with a green light. This indicates that NOAHlink is communicating with the cable attached to the hearing instrument and, therefore, programming the instrument as required.
- Switch off your NOAHlink when you have finished using the device by pressing the button at the front of the unit until all lights no longer display.

(NOTE: If your NOAHlink remains idle for more than 30 minutes, or if it is placed in the charger, it will automatically turn itself off.)

8 Configuring NOAHlink

This chapter describes how to configure your NOAHlink using the NOAHlink Properties window.

To start this window:

- Click  **Start**, point to **Programs**, then **NOAHlink**, and click **NOAHlink Properties**.

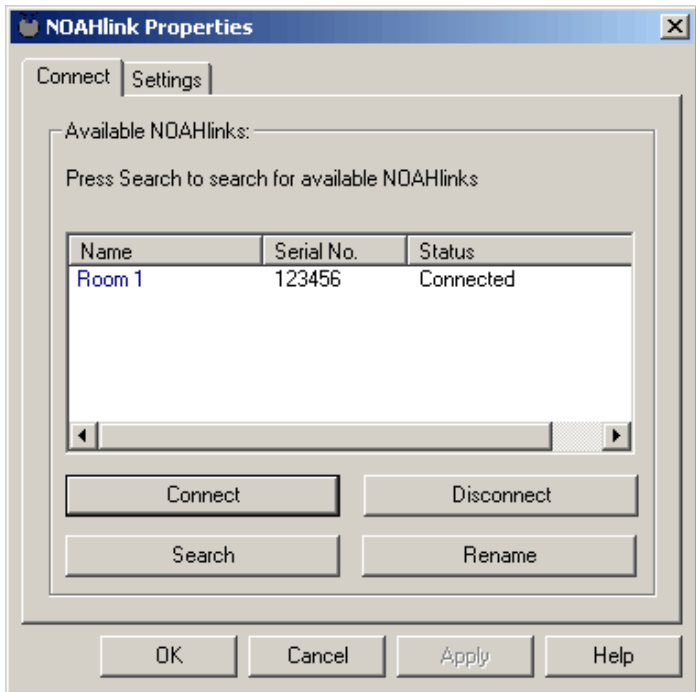
Figure 6: The NOAHlink Properties Window

The following table lists the tabs within the NOAHlink Properties window and the actions they allow you to perform:

TAB LEVEL	TAB	ACTION
General	Connect	<ul style="list-style-type: none"> • Connect to a different NOAHlink • Disconnect your NOAHlink • Search for all NOAHlinks • Rename your NOAHlink
	Settings	<ul style="list-style-type: none"> • Set the battery type • Set a low battery warning • Set your preferred language • Access the advanced tabs
Advanced	Diagnostic	<ul style="list-style-type: none"> • Display errors in NOAHlink and Bluetooth self-tests
	Advanced Settings	<ul style="list-style-type: none"> • Set user preferences • Change the Bluetooth driver • Change the COM port settings
	Performance	<ul style="list-style-type: none"> • Test the transfer rate of data from your PC to your NOAHlink
	Versions	<ul style="list-style-type: none"> • Change the kernel
	Log	<ul style="list-style-type: none"> • Change the location of the log file • View entries in the log • Clear entries in the log • Set the level of information in the log

NOTE: Your NOAHlink must be connected before the Diagnostic, Performance and parts of the Settings tabs can be used. The Advanced tabs can be accessed from the Settings tab. It is necessary for you to configure the General tabs, but you need only deal with the Advanced tabs if you wish to address more specialized issues.

8.1 General Applications

8.1.1 Connecting to NOAHlink

The Connect tab allows you to connect to, disconnect from, search for, or rename a NOAHlink.

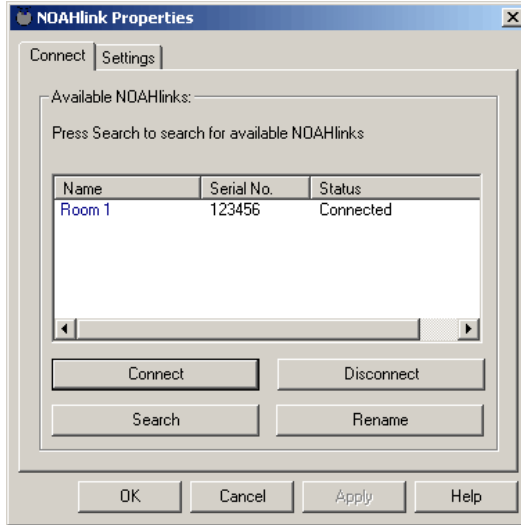
Figure 7: The Connect tab

Each NOAHlink is displayed together with its user-defined alias name, serial number and connection status. The last NOAHlink connected to your PC is highlighted.

The connection status of a NOAHlink can be one of the following:

- In range: Located but actual status unknown
- Available: Located but not connected to any user
- Connected: Located and connected to the NOAHlink Properties program
- Connected by another program: Located and connected to another program on this PC
- In use: Located and connected to another PC
- Known: Not located, but the last NOAHlink connected to your PC

NOTE: When connecting to another NOAHlink, your current NOAHlink connection is automatically disconnected.

You may wish to disconnect your NOAHlink if you require your PC's full power, or your Bluetooth connection, for another task.

The following table describes how to use the Connect tab:

COMMAND	USE	PROCEDURE
Connect	Connect to NOAHlink	<ul style="list-style-type: none"> • Highlight a NOAHlink • Click Connect
Disconnect	Disconnect from NOAHlink	<ul style="list-style-type: none"> • Click Disconnect to disconnect your currently connected NOAHlink
Search	Search for other NOAHlinks	<ul style="list-style-type: none"> • Click Search to search for all NOAHlinks <p>Note: The Search option may take several seconds to complete</p>
Rename	Rename your NOAHlink	<ul style="list-style-type: none"> • Highlight a NOAHlink • Click Rename. This places the cursor at the name, allowing you to overwrite. • Press Return when you have written the new name <p>Tip: It is a good idea to name your NOAHlink after its location or user. Use one of the name labels in the NOAHlink package and attach it to the back of the device.</p>

8.1.2 Setting Power, Battery and Language Options

The Settings tab allows you to set battery options and to create a battery warning. In addition, you can select your preferred language from a list of available NOAH 3 languages.

The **Advanced** button on the Settings tab allows you to access the Diagnostic, Advanced Settings, Performance, Versions and Log tabs.

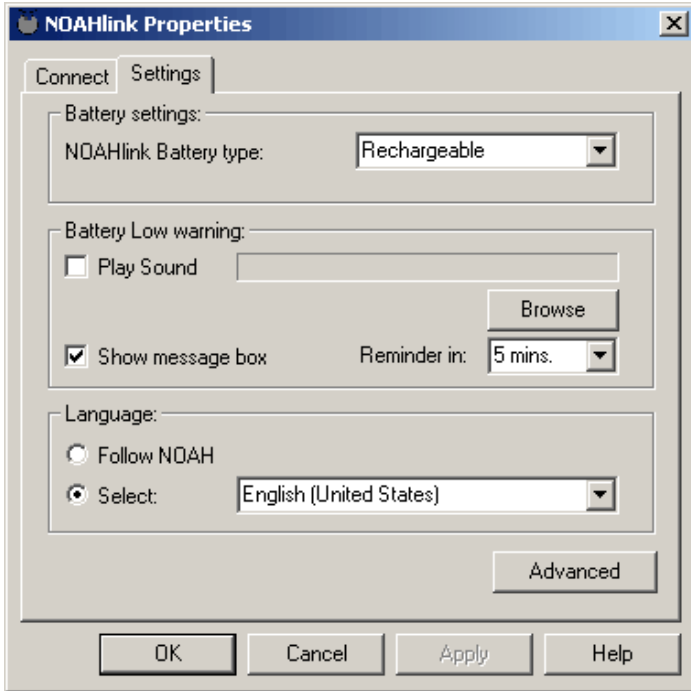
Figure 8: The Settings tab

IMPORTANT: If you do not set the correct battery type, the battery-warning settings will be incorrect. Thus, you may receive a warning even though the battery is working correctly, or you may receive no warning, even though the battery is low.

The following table describes how to use the Settings tab:

SETTING	OPTION	PROCEDURE
Battery	Set battery type	<p>Note: You must be connected to a NOAHlink in order to change its battery settings</p> <ul style="list-style-type: none"> Click the pull-down menu at NOAHlink Battery type to list the battery as Alkaline or Rechargeable <p>Note: The default setting is Rechargeable</p>
	Set 'battery low' warning	<ul style="list-style-type: none"> Click Play Sound or Show message box (or both) to receive a 'low battery' warning Click Browse to choose a warning sound Click the pull-down menu at Reminder in to set the length of time before a reminder is activated (5, 10 or 15 minutes) <p>Note: The default setting shows the message box and plays Chord.wav</p>
Language	Set preferred language	<ul style="list-style-type: none"> Click Follow NOAH to use the current NOAH language <p>OR:</p> <ul style="list-style-type: none"> Click Select and choose your preferred language from the pull-down menu
	Access advanced tabs	<ul style="list-style-type: none"> Click the Advanced button to access the Diagnostic, Advanced Settings, Performance, Versions and Log tabs

8.2 Advanced Applications

8.2.1 Displaying Errors in NOAHlink and Bluetooth

The Diagnostic tab displays errors found in tests on Bluetooth and NOAHlink to check response and other capabilities.

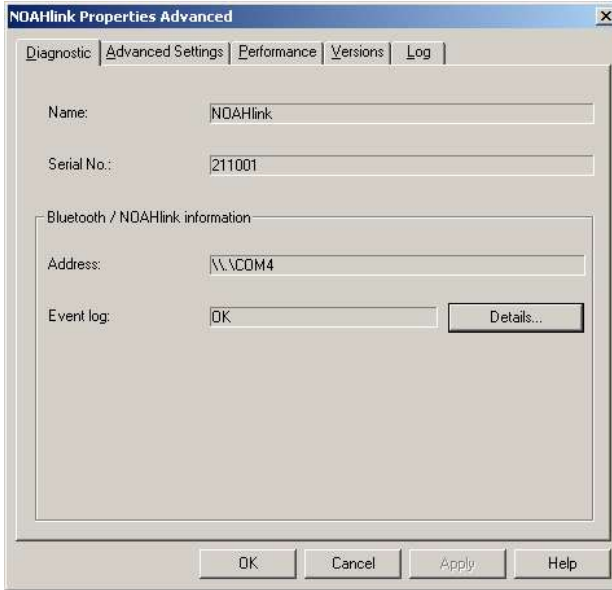
Figure 9: The Diagnostic tab

This tab displays the user-defined name of your NOAHlink, the NOAHlink serial number (printed at the back of the unit) and the Bluetooth device address (or COM port).

The **Event log** is listed as 'OK' or 'Error'.

NOTE: If any errors are listed, these may help your support representative to troubleshoot problems.

The following table describes how to use the Diagnostic tab:

COMMAND	USE	PROCEDURE
Details	List errors if Event log displays 'Error'	<ul style="list-style-type: none"> Click Details to list errors in NOAHlink and Bluetooth tests

8.2.2 Status Settings and User Preferences

The Advanced Settings tab allows you to set user preferences, change the Bluetooth driver should a newer version become available, and change the COM port settings if your system cannot locate NOAHlink.

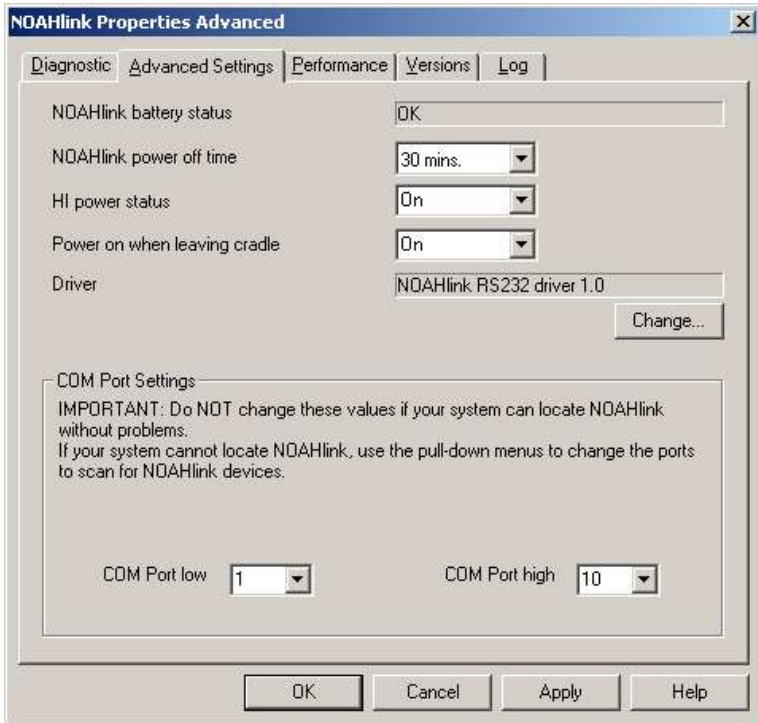
Figure 10: The Advanced Settings tab

The **NOAHlink battery status** is listed as 'OK' or 'Low'.

The following table describes how to use the Advanced Settings tab:

SETTING	OPTION	PROCEDURE
NOAHlink power off time	Set the time length before an idle NOAHlink powers off	<ul style="list-style-type: none"> • Select the length of idle time before NOAHlink powers off <p>OR:</p> <ul style="list-style-type: none"> • Select Off for NOAHlink to remain on indefinitely
HI power status	Turn hearing instruments on automatically with NOAHlink	<ul style="list-style-type: none"> • Select On to turn on hearing instruments automatically when NOAHlink powers on • Select Off for hearing instruments to be turned on through the fitting module
Power on when leaving cradle	Set NOAHlink to power on automatically when removed from its cradle	<ul style="list-style-type: none"> • Select On for a previously powered on NOAHlink to power on automatically when removed from its cradle
Driver	Change the driver	<ul style="list-style-type: none"> • Click Change. A File Menu appears. • Select the driver after browsing for its location • Click OK
COM Port Settings	Scan ports to locate NOAHlink	<p>NOTE: Only select new values if your system CANNOT locate NOAHlink</p> <ul style="list-style-type: none"> • Use the pull-down menus at COM Port low and COM Port high to select new values.

8.2.3 Testing Data Transfer Rates

The Performance tab allows you to test the transfer rate of data from your PC to NOAHlink. You may wish to use this tab if your system is running slowly or unable to retain a connection to NOAHlink.

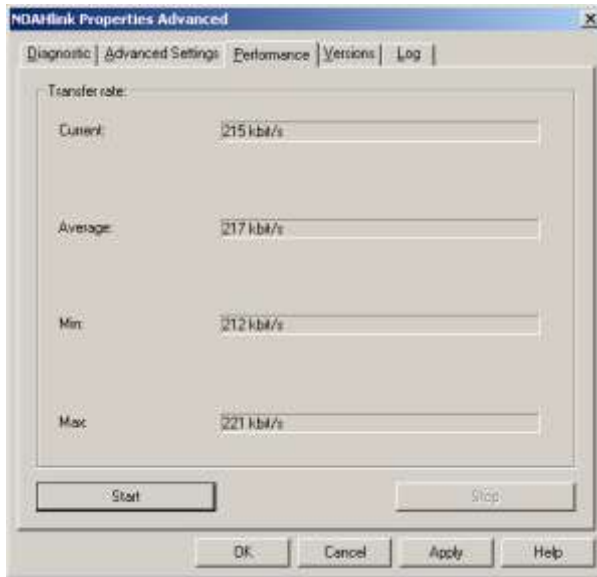
Figure 11: The Performance tab

The following describes how to run the speed test:

- Click **Start** to begin the speed test. (**Note:** Each time you press **Start** the window will be reset.)
- Click **Stop** when the **Min**, **Max** and **Average** fields remain unchanged. (**Note:** The speed test stops automatically if you leave the Performance tab.)

The **Current** figure lists the current transfer rate of data from your PC to NOAHlink; the **Average** figure lists its average transfer rate.

If the **Average** figure is low, causing your device to run very slowly, you may wish to try one of the following:

- move your NOAHlink closer to your PC
- end CPU-consuming programs on your PC which may be responsible for slowing the transfer rate of data
- remove other Bluetooth devices in the area

8.2.4 Version Numbers and Kernel Upgrade

The Versions tab lists software version numbers and allows you to upgrade the kernel should a newer version become available.

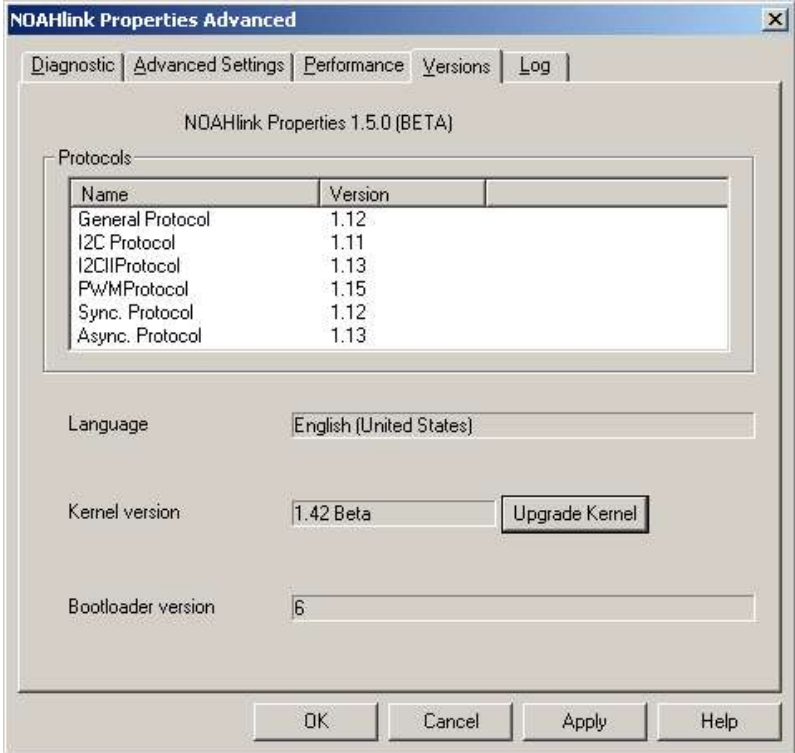
Figure 12: The Versions tab

The following table describes how to use the Versions tab:

COMMAND	USE	PROCEDURE
Upgrade Kernel	Change the kernel	<ul style="list-style-type: none"> • Click Upgrade Kernel to open the Kernel Upgrade window • Use the pull-down menu to select a new kernel • Click Download kernel

8.2.5 Viewing and Setting the Log

The Log tab allows you to view and set NOAHlink's log.

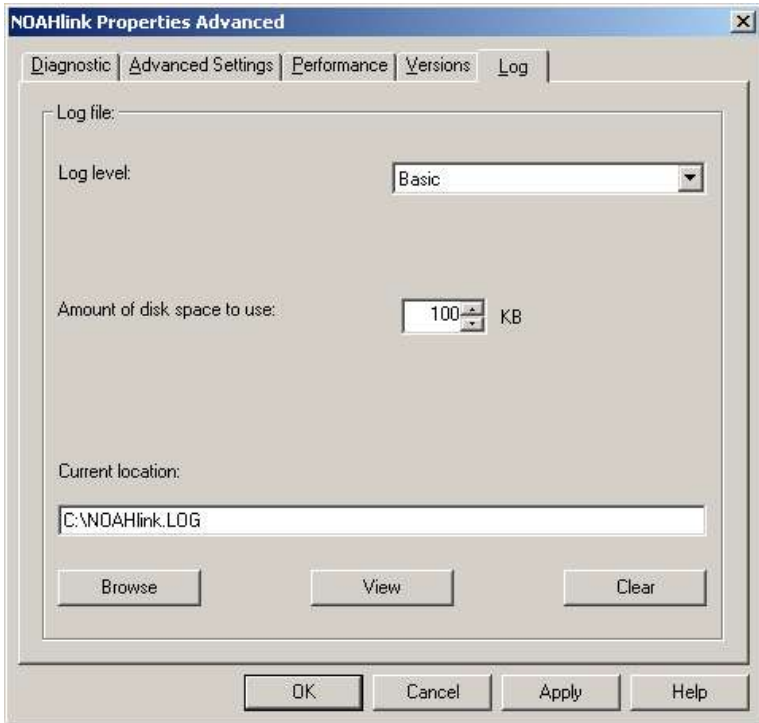
Figure 13: The Log tab

Entries are inserted automatically into the log by NOAHlink, containing the date and time of insertion.

The following table describes how to use the Log tab:

COMMAND	USE	PROCEDURE
Browse	Change current location of log file	<ul style="list-style-type: none"> • Click Browse to locate a new file location OR: Type a new file location in the Current location window
View	View entries in log	<ul style="list-style-type: none"> • Click View to see entries in log
Clear	Clear entries in log	<ul style="list-style-type: none"> • Click Clear to delete all entries in log
	Set form of entries appearing in log	<ul style="list-style-type: none"> • Click the pull-down menu at Log level and set to: <ul style="list-style-type: none"> - None to record no entries - Basic to record errors, load operations and login/logout procedures - Medium to record all Basic entries, methods called in the driver and some diagnostic information - Detailed to record all Medium entries and all other functions <p>Note: The default setting is Basic</p>
	Set amount of disk space for log	<ul style="list-style-type: none"> • Click the pull-down menu at Amount of disk space to use and choose your preferred number of MBs <p>Note: The default setting is 100KB The range is 100KB – 5MB</p>

A NOAHlink Troubleshooting

If you have a support question concerning the installation or operation of NOAHlink, please check through your documentation first. If your problem persists, visit HIMSA's support website (English and German) at www.himsa.com, where we supply a knowledge database with answers to the most frequently asked support questions. If you still cannot find the answers you need, contact your NOAHlink supplier.

Troubleshooting topics:

- *NOAHlink does not power on even though I press the power button:*
 - The battery is inserted incorrectly.
 - The battery needs charging or replacing.
- *NOAHlink powers off and cannot power on even though I press the power button:*
 - The battery needs charging or replacing.
- *The charging indicator does not light up when I place NOAHlink in the charger:*
 - The charger is not connected to the power adaptor, or the power adaptor is not connected to the mains supply.
 - There is no battery in the NOAHlink unit, or the battery is inserted incorrectly.
 - The battery is already fully charged.
- *The discharging indicator does not light up when I press the discharging button:*
 - The battery is already fully discharged.
 - The battery is defective.

B Technical Specifications

This section describes the specifications to which NOAHlink adheres.

B.1 NOAHlink Unit

B.1.1 Battery

Battery Type: Rechargeable (Ni-MH type) AA (R6)

NOTE: Use only rechargeable batteries supplied by HIMSA.

Alkaline AA (R6) 1.5V

Low Battery

Indicator Level: When approximately 45 minutes of battery operating time remain

Estimated

Battery Life: Approximately 20 fittings (each of 45 minutes duration)

B.1.2 Dimensions and Weight

Size: 91mm x 100mm x 31mm (L x W x H)

Net Weight: 115g (excluding battery)

Approx. 141g including battery

B.1.3 Patient Safety

The NOAHlink unit complies with the following standards:

- EN 60601-1, Internal powered, Type BF, IPXO
- UL2601-1 and CAN/CSA-C22.2 NO 601.1-90

To comply with the above standards, the programming cable and the connector to the hearing instrument must meet the following requirements:

- No conductive parts may be accessible when the programming cable is connected to NOAHlink
- The programming cable and the connector must provide double isolation and be able to withstand a dielectric strength test potential of 500 V.

B.2 Charger Unit

B.2.1 Power Supply

Input Voltage: 9V DC \pm 10% / 220 mA

B.2.2 Dimensions and Weight

Size: 125mm x 40mm x 45mm (L x W x H)

Net Weight: 150g (excluding adaptor)

B.3 Power Adaptor

B.3.1 Power Supply

Input Voltage Range: 100 – 240 VAC, 50 – 60 Hz

B.4 Transport and Storage

Temperature: -25°C to +70°C (-13°F to +158°F)

Air Humidity: < 90%, non-condensing

Air Pressure: 500 hPa to 1060 hPa

B.5 Operating Environment

Mode of Operation: Continuous

Temperature: +5°C to +45°C (+41°F to +113°F)

Air Humidity: < 90%, non-condensing

Air Pressure: 700 hPa to 1060 hPa

IMPORTANT: Operation in temperatures below -20°C (-4°F) or above +60°C (+140°F) may cause permanent damage.

B.6 Standard and Optional Accessories

Please refer to Chapter 3 for a list of accessories.

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