noah4

Getting Started

Table of Contents

GETTING STARTED	3
INITIATION	5
CÓMO EMPEZAR.....	7
INICIANDO	9
BEGINN DER ARBEIT	11
AAN DE SLAG.....	13
PREPARAZIONE.....	15
ROZPOCZYNANIE PRACY.....	17
INTRODUKTION.....	19
KOMMA I GÅNG.....	21
KOMME I GANG.....	23
ALOITUS	25
ELSŐ LÉPÉSEK.....	27
BAŞLARKEN.....	29
ZAČÍNAME	31
НАЧАЛО РАБОТЫ	33
スタートガイド.....	35
入门指南	37
入門指南	39
시작 가이드	41

Trademarks Noah is a registered trademark of HIMSA II K/S.

Copyright This document is copyrighted by HIMSA II K/S with all rights reserved. Under copyright laws, this document cannot be reproduced in any form without written permission of HIMSA II K/S.

© 2010 by HIMSA II K/S. All rights reserved.

300-0006

Getting Started

IMPORTANT: If you do not register your Noah 4 license, the Noah software will stop functioning after 45 days. For information on obtaining a license, please contact your Noah distributor.

Preparing for Noah 4 Installation

Before you install Noah 4, you should:

- Make sure your PC(s) meets the minimum Noah 4 requirements
- Determine which modules your Noah 4 installation will need.
- Ensure you have Administrator rights on the PC

Minimum System Requirements

In order to receive satisfactory performance from your Noah 4 application, we recommend that your PC and network server (if applicable) meet or exceed the minimum specifications. (Please note that Noah-compatible modules often have greater minimum requirements than Noah.) For the latest specifications, please visit the Noah Knowledgebase on HIMSA's Support site at www.himsa.com.

Determining the Modules You Will Need

Before installing Noah 4, you should determine which measurement and fitting modules you will need, and have their installation files available. Visit HIMSA's certification site (www.himsa.com) for a list of Noah 4-certified modules.

New Noah 4 user: Contact the hearing instrument and audiological equipment manufacturers with whom you regularly work. They can provide you with Noah modules specifically designed for their products.

Installing Noah & Related Software

1. Download the Noah 4 software to your pc from the Download page at www.himsa.com. Then run the software.
2. Follow the on-screen instructions. After installing, Noah 4 will open automatically. HIMSA recommends enabling password security after logging on – for more details, click on Help in Noah 4.
3. After installing Noah 4, you should install or update your license file – visit HIMSA's support site (www.himsa.com) for further details. If you do not have a Noah license, please contact your Noah distributor.
4. After installing Noah 4, you can begin installing fitting, measurement and product order modules, if necessary.

IMPORTANT: If you have performed a network installation, you will need to install modules on each Noah Client machine.

Information & Support

If you have support questions, please check your Help System. If your problem persists, visit HIMSA's support site (English/German): www.himsa.com. If you still cannot find the answers you need, contact your Noah distributor.

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